

# **Dräger DrugTest® 5000 Drug Screening Device**

No pipetting, no drips, no timing: Collecting an oral fluid sample is quick and easy with the Dräger DrugTest<sup>®</sup> 5000. The collected sample can then be analysed immediately for accurate results on the spot.



### **Benefits**

### Sampling and analysis in one system

The drug analysis system Dräger DrugTest 5000 consists of the sampling system Dräger DrugTest 5000 Test Kit and of the analyzer Dräger DrugTest 5000 Analyzer. This provides you with anything you need for a first assessment on the spot.

### Ready for immediate use, if necessary

The Dräger DrugTest 5000 test kit provides you with a ready-for-use sampling system. You want to check a suspect person? Just remove the cover and hand the test cassette over to the test person.

### Hygienic and painless: Collection without blood samples

The test person moves the top part of the collector briefly back and forth between his cheek and gums until the integrated indicator signals that enough of a sample has been collected. This usually takes less than a minute and never more than four minutes. The test person returns the cassette to you – and you will not come in contact with the saliva sample.

### Blue means sample collection finished

The Dräger DrugTest 5000 automatically indicates the end of the sample collection process: If the collector has collected enough saliva, the indicator becomes blue and the sample collection process is finished. Afterwards you insert the test cassette and cartridge directly into the Dräger DrugTest 5000 Analyzer for evaluation.

### Other advantages of the Dräger DrugTest 5000 test kit:

- high degree of conformity with blood samples and the detection limits valid in Germany (confirmed by a field test by the German Police University)
- extremely low detection limit for THC as the main active ingredient of cannabis
- clear results display
- hygienic sampling and analysis
- high level of acceptance among users and test persons
- easy to use even with safety gloves
- different configurations available (depending on model, several substances can be detected at the same time)
- can be used at temperatures between 4 °C and 40 °C
- can be stored in original package without any deterioration at up to 30 °C

### Immediate evaluation of samples

#### Insert test cassette and cartridge, start evaluation, ready:

The saliva sample can be evaluated immediately after sample collection. Just insert test cassette and cartridge into the analyzer – the evaluation will start immediately.

### **Benefits**

### Easy and safe operation - even under adverse conditions:

Even when it becomes hectic: you control the analyzer with only three keys. The operation is menu-driven. Just follow the plain text instructions.

### Quick and comfortable - results in just a few minutes:

Once the device is put into operation, the analysis will run automatically. This keeps your mind clear and your hands free for attending to your test persons. After only a few minutes, the result will be displayed.

#### Easy-to-read colour display:

The colour display is illuminated. Thus, you stay on top of the situation even under reduced visibility conditions. You can read the display text without any problems even from some distance or from an angle. The Dräger DrugTest 5000 Analyzer also displays the measuring results in clear text. Therefore, misinterpretations do not occur.

### Evaluating samples wherever the situation demands:

Whether you use the sampling system with the 230 V power supply or the 12 V car adapter included, the permanently integrated, rechargeable battery makes you independent of any external power source and the analyzer is ready to use anytime and anywhere.

### Storing 500 measuring results and managing them easily:

It stores the last 500 measuring results including date and time. You can enter further measurement details via keyboard or barcode reader. You can print the results with the Dräger Mobile Printer directly on the spot. To document the data, you can transmit them on the PC via USB.

### 4.5 kg lightweight for transportation:

The analyzer is easy to carry. For transporting the device and its accessories easily and safely, Dräger provides a matching carrying bag or a robust transport case.

### Other advantages of the Dräger DrugTest 5000 Analyzer

- integrated self-test function: no semi-annual calibration necessary
- a variety of accessories (e. g. carrying bag, battery charger or compact keyboard)

### The system detects a variety of substances such as:

- Amphetamines
- Benzodiazepines
- Delta-9-tetrahydrocannabinol (THC)
- Cocaine
- Methamphetamines
- Opiates
- Methadone
- Ketamine

### **Benefits**

### Dräger SSK 5000: Detecting drugs on surfaces or in substances

You want to know on the spot whether drug traces are detectable on a steering wheel or if the contents of a suspicious bag contain drugs? Easily answer such questions by taking a sample with the Dräger SSK 5000 drug sampler and transferring the specimen to the Dräger DrugTest 5000 Test Kit. Then you can analyse the sample with the Dräger DrugTest 5000 Analyzer.

### Accessories



### **Test Kits**

Ergonomically designed for easy handling and ready for immediate use, this innovative test kit consists of one major part, a test cassette complete with oral fluid collector.



### Compact keyboards

- with PS2 interface, dimensions approx. 28.2 x 13.2 x 2.4 cm.
- "QWERTZ" German layout, REF: 8315095
- "QWERTY" English layout, REF: 8315497
- "AZERTY" French layout, REF: 8315142

### Accessories



### **Dräger Mobile Printer**

The Dräger Mobile Printer prints breath alcohol and drug test measurement results without using a dedicated power supply. Operates on 4 AA batteries.

REF: 8319310



### Carrying case for Dräger DrugTest 5000

REF: 8322675



### Hard-shell carriying case

REF: 8319925



### Barcode scanner

REF: AG02491

### Related Products



### Dräger DCD 5000

Simple, fast and secure detection of intoxicants and drugs: The Dräger DCD 5000 does not only enable an easy collection of saliva, you can also store the samples and send them to the laboratory for analysis – without the sample being falsified along the way.



### Dräger Alcotest® 6820

Dräger Alcotest® 6820 is the logical successor to the successful Alcotest® 6810. The compact hand-held measuring instrument Alcotest® is quickly ready for use and is adaptable to international guidelines. With an even more sturdy enclosure and the proven Dräger sensor, Alcotest® 6820 is a reliable partner for breath alcohol analyses.



### Dräger Alcotest® 7510

The Dräger Alcotest® 7510 is a compact and robust handheld breath alcohol measuring instrument, especially designed for advanced screening applications. Ideal for use by police as well as in commerce and industry as an alcohol screening device, it can also be used for evidential testing purposes, subject to relevant local and market sector approvals.

0 0 0

### **Technical Data**

| Operating voltage      | 12 V direct voltage (via supplied power pack)                      |  |
|------------------------|--|--|
| Power input            | normally 3 A   |  |
| Dimensions (W × H × D) | 200 × 260 × 250 mm   |  |
| Weight                 | 4.5 kg   |  |
| Operating range        | Operation: 4 °C to 40 °C   |  |
|                        | Storage / transportation: -20 °C to +60 °C                         |  |
|                        | relative humidity 5 to 95 % (non-condensing)                       |  |
| Ports                  | optical for printer, PS/2, USB Slave                               |  |
| Time of measurement    | depending on used test kit (typically < 9 minutes)                 |  |
|                        | optional: Analysis in < 5 minutes by increasing the THC cut-off to |  |
|                        | 25 ng/ml (after updating the device software)                      |  |
| Storage capacity       | 500 records or measurements  |  |

## Ordering Information

| Packaging unit              | Order No.   |
|-----------------------------|---|
| 1 analyzer incl. power pack | 83 19 900   |
| 20 pieces                   | 83 19 830   |
|                             |   |
| 20 pieces                   | 83 21 280   |
|                             |   |
| 20 pieces                   | 83 19 990   |
| 20 pieces                   | 83 23 156   |
|                             |   |
| 20 pieces                   | 83 23 646**   |
|                             |   |
| 20 pieces                   | 83 20 490   |
| 1                           | 83 15 095   |
|                             |   |
| 1                           | 83 15 497   |
|                             |   |
| 1                           | 83 15 142   |
|                             |   |
| 1                           | AG 02 491   |
| 1                           | 83 19 310   |
| 1                           | 83 22 675   |
|                             |   |
| 1                           | 83 19 925   |
|                             |   |
|                             | 1 analyzer incl. power pack 20 pieces 20 pieces 20 pieces 20 pieces 20 pieces 1 1 1 1 1 1 1 1 |

<sup>\*</sup> COC: cocaine - \*OPI: opiates - \*BENZO: benzodiazepines - \*THC: cannabis - \*AMP: amphetamine - \*MAMP: methamphetamines

<sup>- \*</sup>MTD: methadone - \*KET: Ketamine

<sup>\*\*</sup> Sold only within the European Union

### Notes

### CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA Moislinger Allee 53–55 23558 Lübeck, Germany www.draeger.com

### Manufacturer:

Dräger Safety AG & Co. KGaA Revalstraße 1 23560 Lübeck, Germany

# REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Safety AG & Co. KGaA Revalstraße 1 23560 Lübeck, Germany Tel +49 451 882 0 Fax +49 451 882 2080 info@draeger.com

### REGION EUROPE SOUTH

Dräger Safety France SAS 25, rue Georges Besse 92182 Antony Cedex Tel +33 1 46 11 56 00 Fax +33 1 40 96 97 20 europe.south@draeger.com

### REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA Branch Office P.O. Box 505108 Dubai, United Arab Emirates Tel +971 4 4294 600 Fax +971 4 4294 699 contactuae@draeger.com

#### REGION ASIA PACIFIC

Draeger Safety Asia Pte Ltd 25 International Business Park #04-20/21 German Centre Singapore 609916 Tel +65 6308 9400 Fax +65 6308 9401 asia.pacific@draeger.com

### REGION CENTRAL AND SOUTH AMERICA

Dräger Panama S. de R.L. Complejo Business Park, V tower, 10th floor Panama City Tel +507 377 9100 Fax +507 377 9130 contactcsa@draeger.com

Locate your Regional Sales Representative at: www.draeger.com/contact

